USER'S MANUAL

IceBand[®]

by MD R. Ihrman Patent Pend.

Cooling and compression bandage

Article: Face

Article no: 4002

For single patient use only

Cooling agent: Soft ice

Contains: Wrap and 2 cooling elements

Efficient cooling time: up to 1 hour

Durability: approx. 30 applications

Users manual: please see inside

Patent Pend.

All rights reserved

IceBand[®] is a registered trademark owned by IB MEDICAL AB


CE class 1

Non sterile

Manufacturer: IB MEDICAL AB, Sweden

www.ibmedical.se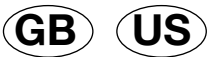

IceBand® Face is a cooling and compression bandage specially developed for rehabilitation and pain relief associated with oral and facial surgeries or procedures.

Reading this user's Manual will enable you to obtain the maximum therapeutic effect.

IceBand Face should be applied without delay after surgery/procedure to obtain the maximum effect.

IceBand can facilitate postoperative rehabilitation and shorten your recovery.

USER'S MANUAL

- Place the cooling elements in the freezer (freeze to ice within about 3 hours).
- The face bandage is made so that it can be used either on one side of the face or on both sides.
- Remove 1 cooling element if you have only operated on one side and insert it in the bandage pocket (fig.1) Keep the second cooling element in the freezer and use it as an exchange when the first has melted.
- Attach the bandage with Velcro straps over the head and behind the neck.
- The bandage should sit comfortably and with good compression for full effect.
- Allow the bandage to work for 45-60 minutes for maximum pain relief and detumescence. We recommend using IceBand for repeated intervals of approx. 45-60 minutes each time.
After the 45-60 minutes remove the IceBand for at least one hour. This allows the operated area to "rest". Repeat the process several times.
- The IceBand should be removed when the cooling element has melted and the cooling effect has come to an end.
- Repeat the treatment 4-6 times a day for the first few days after surgery and then as needed.

IceBand Face should be used only on the recommendation of your dentist. Discontinue treatment and consult with your dentist if you get more sore or if you feel unsafe.

IceBand Face is for Single Patient use only.

CONTRAINDICATIONS: Cold should not be used if you suffer from cold allergy, Raynaud's phenomenon or cryoglobulinaemia.

CAUTION: Never place the cooling elements directly on the skin. If you are unsure consult your doctor.

IceBand® was developed in collaboration with MD Robert Ihrman at Orthopedic clinic, County Hospital Ryhov in Jönköping, Sweden.

Durability: Holds for approx. 30 times use.

Cleaning: The cooling elements can be disinfected and washed in a dishwasher for max. 65 C. The fabric should not be washed.

Waste: Contains water, polyethylene, polypropylene and is handled as usual waste. In combustion, carbon dioxide and water are formed.

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Patent Pend. All rights reserved. Non sterile
IceBand® is a registered trademark owned by IB MEDICAL AB
IB Medical AB,