USER MANUAL

IceBand[®]

by MD R. Ihrman Patent Pend.

Cooling – compression bandage

Article: IceBand Shoulder

Article no: IB 4006 U

For single patient use only

Cooling agent: water. PCE™ technology

Contains: wrap and 2 x 2 cooling elements

Efficient cooling time: up to 1 hour

Durability: approx. 30 applications

Users manual: please see inside

Patent Pend.





All rights reserved

IceBand[®] is a registered trademark owned by IB MEDICAL AB

CE CE class 1

Manufacturer: IB MEDICAL AB, Sweden

icebandmedical.com

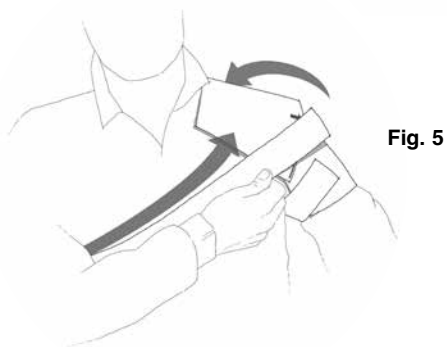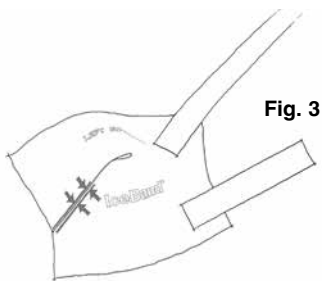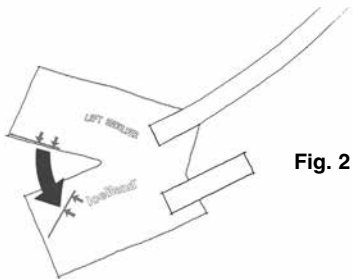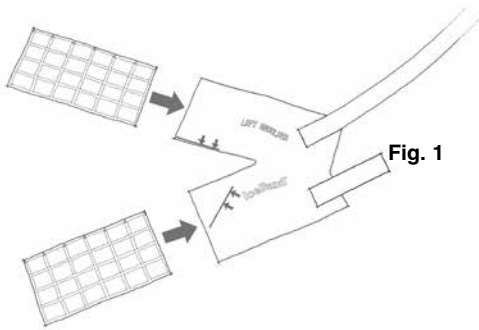IceBand® is a cold therapy bandage intended for use after orthopedic surgery and during the rehabilitation period after. IceBand® can also be used in case of acute injury while waiting for medical attention.

Cooling and compression in combination relieves pain and reduces swelling.

Reading this Users' Manual will enable you to obtain the maximum therapeutic effect.

IceBand® should be applied without delay after surgery and can also be used after physical therapy and rehabilitation.

IceBand® can facilitate postoperative rehabilitation and shorten your recovery.

Cooling the shoulder has an anti-inflammatory effect that can reduce swelling and pain.

IceBand® can also be used to help reduce pain and swelling following an acute injury, while you are awaiting medical attention.

IceBand® Includes 1 shoulder wrap and 2 sets of 2 cooling elements.

The cooling agent is pure water without any additives, using PCE™ technology. (Phase Changing Element)

USERS' MANUAL

- Put the cooling elements in a freezer (within approx. 3 hours the water will have frozen to ice).
- Insert one cooling element in each of the two pockets of the shoulder wrap and apply the wrap to your shoulder. Keep the remaining cooling elements in the freezer and use these as replacements when the first two have melted.
- Tighten the Velcro fastenings securely to provide desired compression.
- Allow the IceBand® to work for 45-60 minutes for maximum effect. We recommend using IceBand for repeated intervals of approx. 45-60 minutes each time. After the 45-60 minutes, take off the IceBand for at least one hour. This enables the operated area to "rest". Repeat the process several times.
- The IceBand® should be removed when the ice has all melted and the cooling effect has come to an end.
- Repeat the treatment several times postoperatively and after physical therapy and rehabilitation sessions as instructed by your healthcare professional.

CAUTION! The cooling elements shall never be applied directly to the skin. Only to be used inside the wrap. Patients' skin should be observed frequently due to individual sensitivity and susceptibility to cool.

Contact your physician if the treatments lead to more pain or if you are uncertain about any aspect of its use. Cooling products shall be handled with care and only be used in accordance with instructions from a physician.

CONTRAINDICATIONS: IceBand® should not be used if you suffer from cold allergy, Raynauds phenomenon or cryoglobulinaemia. If you are uncertain please ask your physician.

For single patient use only

Each IceBand® can be used approximately 30 times.

CLEANING: The cooling elements can be disinfected using a surface disinfectant and can be washed in a dishwasher at max. 65 degrees Celsius / 150 Fahrenheit. The fabric shall not be washed.

DISPOSAL: Consists of water, polyethylene, and polypropylene and can be exposed of as normal household waste. When incinerated it will turn into CO² and water.

IceBand® was developed in co-operation with: Robert Ihrman, MD, Orthopaedic Consultant, Orthopaedic Clinic, Ryhov County Hospital in Sweden.

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IceBand® is a registered trademark owned by IB MEDICAL AB
IB Medical AB